

# Mucus Clearance: Flutter

**FLUTTER** )))))))


The Flutter is a small, lightweight mucus clearance device that requires no power and can be used anywhere and at anytime.

#### Using the Flutter can help to:

- Loosen trapped mucus
- Keep the airways open

#### The Flutter is suitable for the sufferers of:

- Chronic Bronchitis
- Bronchiectasis
- Emphysema
- Asthma
- Cystic Fibrosis

#### How does it work?

- User exhales into the device
- This causes a ball bearing inside the casing to vibrate
- Vibrations travel back into the lungs
- These vibrations loosen the mucus in the airways
- Gentle coughing then removes the mucus
- Sessions normally last between 5 and 15 minutes


[www.clement-clarke.com](http://www.clement-clarke.com)

**CLEMENT CLARKE  
INTERNATIONAL**

Engineered for Life

**TOTAL RESPIRATORY SOLUTIONS**

 Purchase this on our E-Shop at [www.clement-clarke.com/shop](http://www.clement-clarke.com/shop)

## Technical Specifications

Dimensions: 15 x 5.2 x 3.8cm

Weight: 60g

## Part Codes


Flutter 3102000

## Mucus Clearance

The Flutter is commonly used for sessions lasting between 5 and 15 minutes.

The Flutter is not suitable if you suffer from the following:

- Pneumothorax
- Severe Tuberculosis
- Haemoptysis (Coughing up blood)

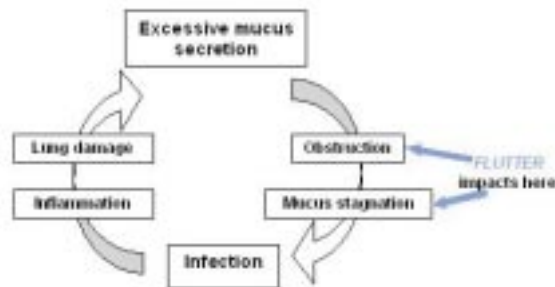

**Clement Clarke** are well established in offering accurate, precise and dependable products for the diagnosis, monitoring and treatment of respiratory disease

Clement Clarke continues to develop broad product ranges to strengthen our portfolio in **Asthma Management, COPD, Smoking Cessation, Cystic Fibrosis and Allergic Rhinitis** in providing the total respiratory solution.